


## Unimount™ Bracket Installation

Be sure the grey knob is loose. Turn counter-clockwise until the tongue is centered in the slot.

Insert the two fixed legs of the Unimount™ Bracket into to the equipment channel

Turn the bracket so that it the legs and the tongue are inserted into the equipment channel

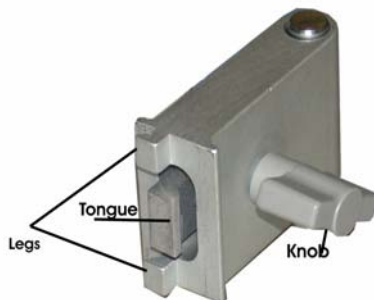

Tighten the grey knob [clockwise] to hold the Unimount™ Bracket in position.




Note: Do not over-tighten



## Basket Installation

Insert the fixed legs of the mounting bracket into the equipment channel

Turn the Unimount bracket to engage the tongue into the equipment channel and tighten the grey knob.

 Note: Do not over-tighten



## Sphygmomanometer Installation

Insert the peg at the bottom of the sphygmomanometer mount into the Unimount™ Bracket



For installation of the tubing and cuff please see the instructions for the sphygmomanometer



## Vacuum Bottle Slide Installation

To install the vacuum bottle slide, you will need a Philips head screwdriver.

Bring the vacuum bottle slide to the vertical track to which it will be installed. On the back of the slide you will see a diamond shaped nut and a round peg. The diamond should be loose. Turn the diamond nut as shown on in the photo.

Place the slide against the mounting track with the diamond nut fitting into the track. As you begin to tighten the screw on the front of the diamond nut it will first turn the diamond nut in the channel so that it locks into the track.

Adjust the slide to the desired position

Tighten the screw



## Collection Bottle Installation

Assemble the collection bottle by snapping the white top into the clear container.

Slide the holder into the vacuum bottle slide

Place the assembled collection bottle into the ring holder

